

## BACKGROUND


- Bacterial strain typing using whole genome sequencing (WGS) plays a key role in nosocomial outbreak investigations
- While WGS is an improvement over traditional typing methods due to higher resolution, manual WGS workflows can be time-consuming

## OBJECTIVES


- Compare a manual WGS method to an automated workflow using the ClearDx™ platform (Clear Labs) for use in bacterial strain typing in terms of comparability of results, cost, and time.

## METHODS

**FIGURE 1: MANUAL WGS METHOD WORKFLOW**

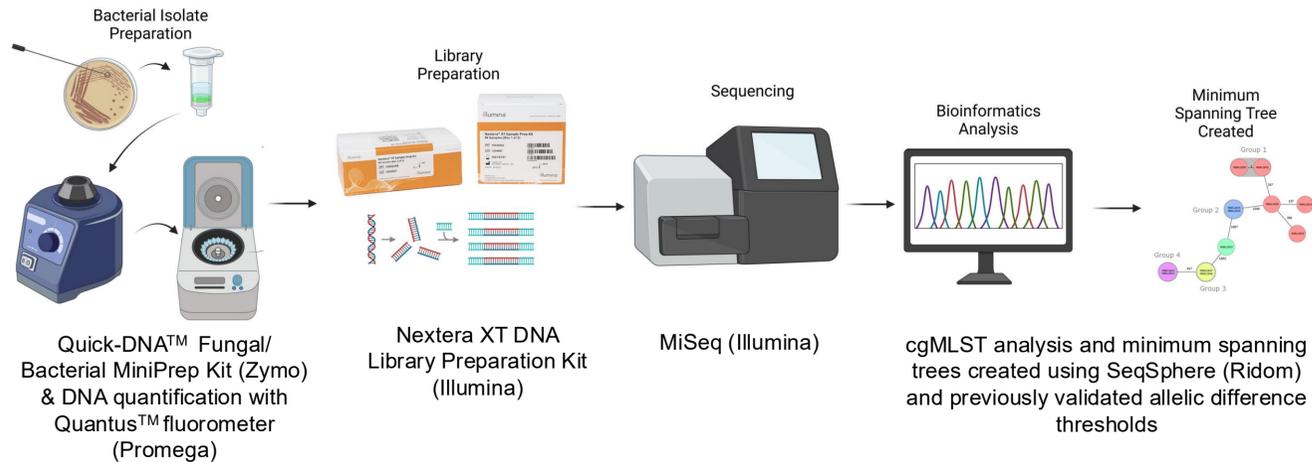

**FIGURE 2: AUTOMATED WGS METHOD WORKFLOW**

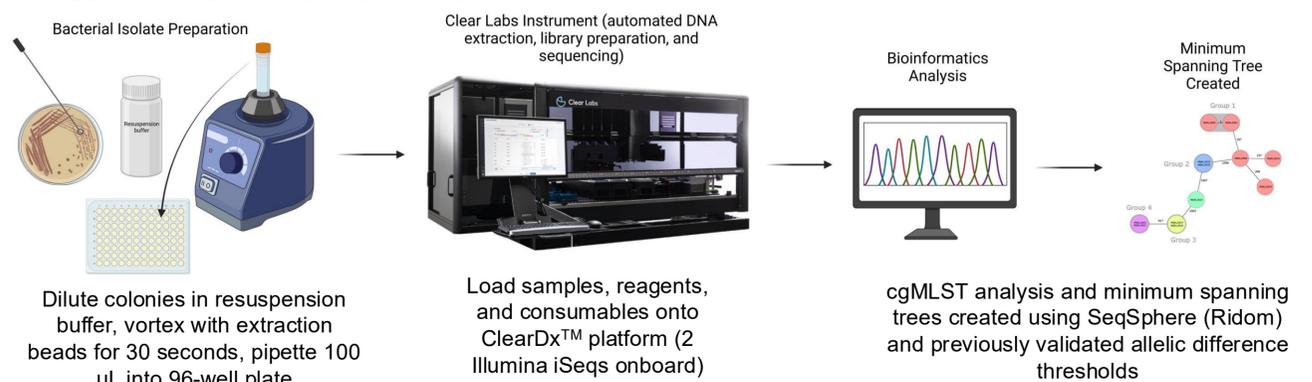


## RESULTS

**TABLE 1: BACTERIAL ISOLATES SEQUENCED**

Organism	Number of Isolates
<i>Acinetobacter baumannii</i>	8
<i>Campylobacter jejuni/coli</i>	16
<i>Clostridioides difficile</i>	8
<i>Cutibacterium acnes</i>	12
<i>Enterobacter cloacae</i> complex	4
<i>Enterococcus faecalis</i>	7
<i>Enterococcus faecium</i>	12
<i>Escherichia coli</i>	8
<i>Klebsiella pneumoniae</i>	8
<i>Legionella pneumophila</i>	8
<i>Pseudomonas aeruginosa</i>	10
<i>Serratia marcescens</i>	8
<i>Staphylococcus aureus</i>	72
<i>Staphylococcus epidermidis</i>	12
<i>Staphylococcus lugdunensis</i>	12
<i>Streptococcus agalactiae</i>	9
<i>Streptococcus pyogenes</i>	12

**FIGURE 3: COST PER TEST BY BATCH SIZE**

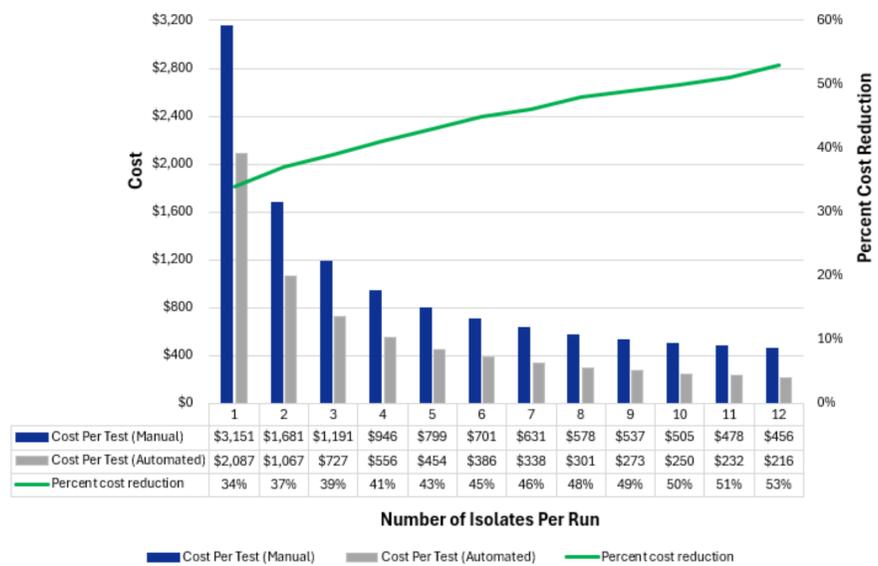

Cost differences were compared by calculating labor, reagents, consumables and standard overhead for each method. Automation reduced labor costs by 93% and overall costs from 34 to 53% for run sizes of one to 12 isolates, respectively

**TABLE 2: AVERAGE TIME PER TESTING STEP**

Step	Manual WGS Method		Automated WGS Method	
	Hands-on Time	Additional Automated Analytical Time	Hands-on Time	Additional Automated Analytical Time
Sample Preparation	30 minutes	Not applicable	30 minutes	Not applicable
Instrument Setup	Not applicable	Not applicable	30 minutes	Not applicable
Library Preparation	3 hours	Not applicable	Not applicable	7 hours
Sequencing	30 minutes	39 to 42 hours	Not applicable	19 hours
Bioinformatics Analysis	1 hour	Not applicable	1 hour	Not applicable
<b>Total</b>	<b>5 hours</b>	<b>39 to 42 hours</b>	<b>2 hours</b>	<b>26 hours</b>
<b>Total Time</b>	<b>44 to 47 hours</b>		<b>28 hours</b>	

## DISCUSSION

- 222/224 (99%) isolates that generated sequences showed concordant groupings between automated and manual methods
- Mantel Spearman correlation of paired distance matrices showed statistically similar results between automated and manual methods
- Use of Clear Dx™ platform decreased turnaround time from 44 to 47 hours with manual method to average of 28 hours with automation, eliminating 3 hours of manual work
- Cost analysis predicted cost savings of 34 to 53% compared to manual method

## CONCLUSIONS

- Integration of automation into a bacterial WGS workflow yielded nearly equivalent results to a more manual workflow with improved turnaround time and projected cost savings

## ACKNOWLEDGEMENTS

The authors would like to thank the Bacteriology Laboratory staff at Mayo Clinic, Rochester, MN for performing the manual sequencing method and the Process Innovation Through Automation Laboratory at Mayo Clinic, Rochester, MN for the use of laboratory space. We thank the Minnesota Department of Health, the Antibacterial Resistance Leadership Group, Dr. Madhusudan Grover (Mayo Clinic), and Dr. Faten El Sayed (Assistance Publique - Hôpitaux de Paris, Université Paris Saclay) for providing the isolates studied. This study was partially funded by Clear Labs, Inc. through an instrument evaluation agreement; Clear Labs, Inc. provided the instrument and select reagents and consumables.